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Módulo

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Description and intended use:

Dental guard designed to protect the patient's upper or lower teeth during medical procedures, preventing damage to the teeth or the accidental entry of fragments into the trachea or lungs. Manufactured with a tough outer shell that acts as a barrier between the teeth and medical instruments. With an inner layer of shock-absorbing material to cushion impacts and provide extra protection. Includes a safety cord to facilitate the removal of the guard when necessary

Indications:

For use in endoscopic surgery and procedures requiring the use of a laryngoscope blade. Dental protection. Prevention of dental fracture.

Contraindications:

- Non-sterile product. Do not use when sterile conditions are required.
- Do not use in children or patients with very small dental arch sizes.
- Not recommended for use in patients with dentures.
- Do not use to treat dental conditions such as bruxism.
- Do not use in patients with advanced dental mobility.
- Do not reuse the product; single patient use only.

Warnings:

- For use only by appropriately trained medical personnel.
- Do not use the product for more than 8 continuous hours.
- Do not use the product if the container is open, broken or appears damaged.
- Remove all removable prostheses or dentures from the patient's mouth prior to use.
- The safety cord should remain outside the patient's mouth to facilitate removal of the guard after use.
- The use of the dental guard may make access to the oral cavity more difficult by reducing the space available.
- Do not use the guard without the inner shock absorber.
- Do not use the guard without the safety cord.
- Reusing this product may result in cross-infection risks for the patient.
- Carry out a final count of the items in the set to ensure they have not been inadvertently left in the patient (see Fig. 1).

Note: All serious incidents involving this product must be reported to the manufacturer and to the competent authority in the user and/or patient's Member State.

Instructions for use:

- Open the bag immediately before use without using any pointed or sharp objects that could damage the product.
- Extend the safety cord. Fit the flexible polyethylene shell to the patient's teeth and press gently so that the guard conforms to their shape (see Fig. 2).
- The safety cord should remain outside the patient's mouth to allow subsequent removal of the guard.
- After use, remove the guard from the patient's mouth with the help of the safety cord.

Disposal of the product:

After use the product is biologically contaminated and must be treated according to existing protocols for biologically contaminated materials

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